PHILIPS

AVENT

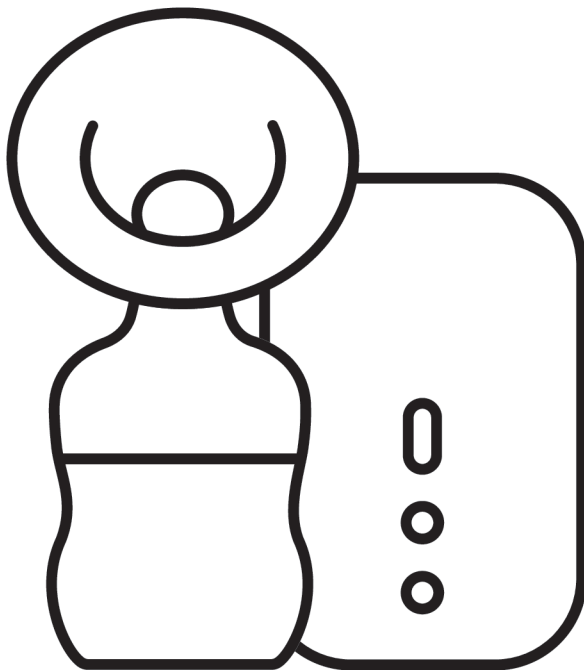
Single/Double
Electric Breast
Pump

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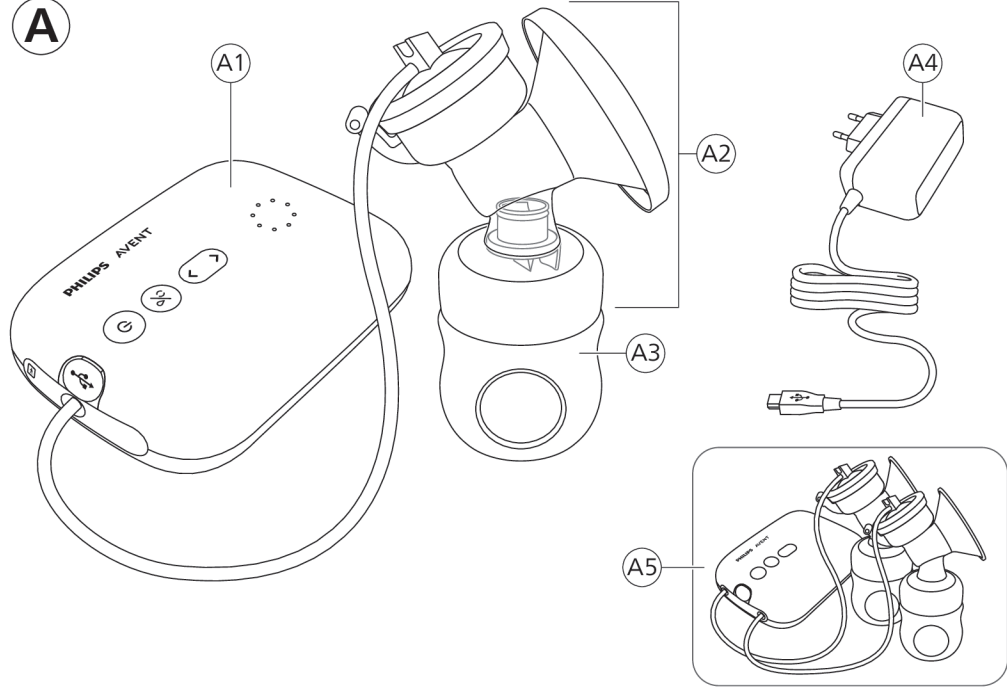
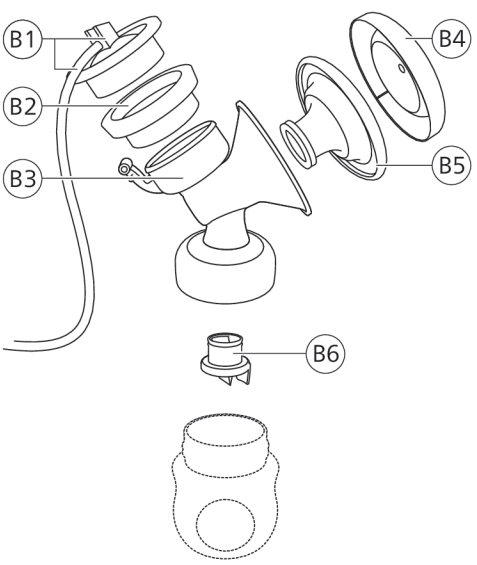
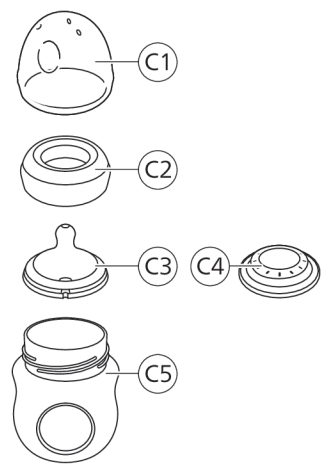
Contents

Introduction _____	6
General description _____	7
Intended use _____	7
Important safety information _____	8
Before first use _____	10
Cleaning and disinfecting _____	10
Using the breast pump _____	13
Feeding breast milk with the bottle _____	19
Compatibility _____	21
Ordering accessories and spare parts _____	21
Recycling _____	21
Support _____	21
Troubleshooting _____	22
Supplemental information _____	23
Storage conditions _____	23
Electromagnetic fields (EMF) _____	24
Technical information _____	24
Explanation of symbols _____	25

Introduction

Congratulations on your purchase and welcome to Philips Avent! To fully benefit from the support that Philips Avent offers, register your breast pump at www.philips.com/mybreastpump.

The Philips Avent single/double electric breast pump Advanced is inspired by babies and their natural suction movement. The soft silicone cushion stimulates the nipple with gentle massaging movements, helping you to express milk comfortably and effectively. The combination of suction and nipple stimulation triggers fast milk flow. The no-lean-forward design enables you to sit in a relaxed, upright position while expressing.

Healthcare professionals and the World Health Organization recommend to exclusively feed children with breast milk during the first six months of life and to continue giving them breast milk for two years next to offering additional nutritious foods. This is because your breast milk is specially adapted to your baby's needs and helps to protect your baby against infection and allergies. To breastfeed longer, you can express and store your milk so that your baby can still enjoy the benefits of it, even if you cannot be there to provide it yourself. As the pump is compact, quiet and discrete to use, you can take it with you anywhere, allowing you to express milk at your own convenience and to maintain your milk supply.

If breastfeeding goes well, it is advisable (unless recommended otherwise by your healthcare professional) to wait until your milk supply and breastfeeding schedule are established (normally at least 2 to 4 weeks after giving birth) before you start expressing.

General description

i Note: The double breast pump comes with two pieces of the items shown in Figures B and C.

Product description (Fig. A)

A1 Motor unit

A2 Breast pump kit

A3 Bottle

A4 Adapter*

A5 Double breast pump

* The adapter supplied may vary for different countries.

Breast pump kit (Fig. B)

B1 Silicone tube and cap

B2 Silicone diaphragm

B3 Pump body

B4 Cover

B5 Cushion

B6 White valve

Philips Avent Natural bottle (Fig. C)

C1 Bottle cap

C2 Screw ring

C3 Teat

C4 Sealing disc

C5 Bottle

Accessories

There are multiple versions of this breast pump package, which all come with a different combination of the following Philips Avent accessories:

- Breast pads
- Travel bag
- Insulation pouch
- Cleaning brush
- Storage cups with adapter*
- Breast milk storage bags*
- Natural bottles

* For these accessories, a separate user manual is provided.

Intended use

The Philips Avent single/double electric breast pump Advanced is intended to express and collect milk from the breast of a lactating woman. The device is intended for a single user.

Important safety information

Read this user manual carefully before you use the breast pump and save it for future reference.

Warning

- Never use the breast pump while you are pregnant, as pumping can induce labor.

Warnings to avoid choking, strangulation and injury:

- Do not allow children or pets to play with the motor unit, the adapter, packaging materials or accessories.
- Always unplug the breast pump after use. Store the breast pump when not in use.
- This breast pump is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge. Such persons can only use this breast pump if they are supervised by or have received proper instruction concerning use of the appliance by a person responsible for their safety.

Warning to prevent transmitting infectious diseases:

- If you are a mother who is infected with Hepatitis B, Hepatitis C or Human Immunodeficiency Virus (HIV), expressing breast milk will not reduce or remove the risk of transmitting the virus to your baby through your breast milk.

Warnings to avoid electric shock:

- Inspect the breast pump, including the adapter, for signs of damage before each use. Do not use the breast pump if the adapter or plug is damaged, if it does not work properly or if it has been dropped or submerged in water.
- Only use the adapter (S009AHz050yyyy) provided with this breast pump.
- Always ensure that it is easy to disconnect the device from the mains in case of malfunction. The adapter is considered the disconnection device.

Warnings to avoid contamination and to ensure hygiene:

- For hygienic reasons, the breast pump is only intended for repeated use by a single user.
- Clean and disinfect all parts that come into contact with the breast and breast milk before first use and after each use.
- Do not use antibacterial or abrasive cleaning agents to clean the breast pump parts as this may cause damage.
- Wash your hands thoroughly with soap and water before you touch breast pump parts and breasts to prevent contamination. Avoid touching the inside of containers or lids.

Warnings to avoid breast and nipple problems including pain:

- Do not use the breast pump if the silicone diaphragm appears to be damaged or broken. See 'Ordering accessories and spare parts' for information on how to obtain replacement parts.
- Only use accessories and parts recommended by Philips Avent.
- No modification of the breast pump is allowed. If you do so, your warranty becomes invalid.
- Never use the breast pump while you are sleepy or drowsy to avoid lack of attention during use.
- Always switch off the breast pump before you remove the pump body from your breast, in order to release the vacuum.
- If the vacuum is uncomfortable or causes pain, switch off and remove the breast pump from your breast.
- Do not continue pumping for more than five minutes at a time if you do not succeed in expressing any milk. Try to express at another time during the day.

- If the process becomes very uncomfortable or painful, stop using the breast pump and consult your healthcare professional.

Warnings to prevent damage and malfunction of the breast pump:

- Portable radio frequency (RF) communications equipment (including mobile phones and peripherals such as antenna cables and external antennas) should not be used closer than 30 cm (12 in) to any part of the breast pump, including adapter. This could have a negative effect on the performance of the breast pump.
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation.

Caution

**Cautions to prevent damage to and malfunctioning of the breast pump:**

- Prevent the adapter and motor unit from coming into contact with water.
- Never put the motor unit or the adapter in water or in a dishwasher, as this causes permanent damage to these parts.
- Keep the adapter and the silicone tubes away from heated surfaces to avoid overheating and deformation of these parts.

Before first use

Disassemble, clean and disinfect all parts that come into contact with the breast and breast milk before you use the breast pump for the first time and after each use.

Cleaning and disinfecting

Overview

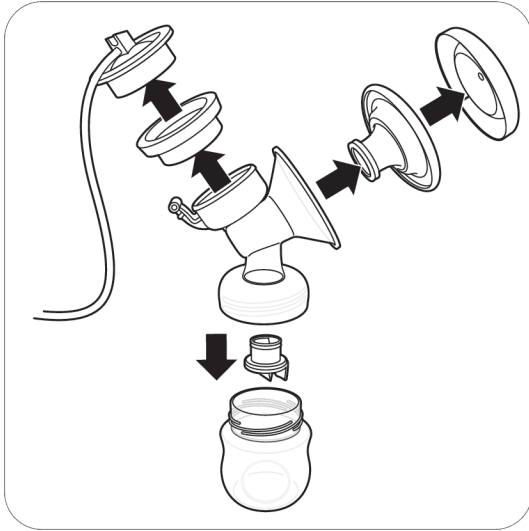
Clean and disinfect the parts that come into contact with the breast and breast milk as described below:

	When	How
	Clean and disinfect before first use and after every use.	Disassemble all parts and then clean them as described in chapter 'Cleaning' and disinfect them as described in chapter 'Disinfecting'.
		

The following parts do not come into contact with the breast and breast milk; clean them as described below:

	When	How
	Clean when needed.	Wipe with a clean, damp cloth with water and mild detergent.

Step 1: Disassembling

Disassemble the breast pump, bottle and storage cup completely. Also remove the white valve from the pump body (Fig. B3).

Step 2: Cleaning

The parts that come into contact with the breast and breast milk can be cleaned manually or in the dishwasher.

⚠ Warning: Do not use antibacterial or abrasive cleaning agents to clean the breast pump parts as this may cause damage.

⚠ Caution: Never put the motor unit or the adapter in water or in a dishwasher, as this causes permanent damage to these parts.

⚠ Caution: Be careful when you remove and clean the white valve. If it gets damaged, your breast pump does not function properly. To remove the white valve, gently pull at the ribbed tab on the side of the valve. To clean the white valve, rub it gently between your fingers in warm water with some dishwashing liquid. Do not insert objects into the white valve, as this may cause damage.

Step 2A: Manual cleaning

Supplies needed:

- Mild dishwashing liquid
- Drinking-quality water
- Soft, clean brush
- Clean tea towel or drying rack
- Clean sink or bowl

1. Rinse all parts under a running tap with lukewarm water.

2. Soak all parts for 5 minutes in warm water with some mild dishwashing liquid.

3. Clean all parts with a cleaning brush in warm water with some mild dishwashing liquid. If you use a brush to clean the tip of the feeding teat, clean it as carefully as possible to avoid damage.

4. Thoroughly rinse all parts under a running tap with cold clear water.

5. Leave all parts to air-dry on a clean tea towel or drying rack.

Step 2B: Cleaning in the dishwasher

Supplies needed:

- Mild dishwashing liquid or a dishwashing tablet
- Drinking-quality water

i Note: Food colorings may discolor parts.

1. Place all parts on the top rack of the dishwasher.

2. Put dishwashing liquid or a tablet in the machine and run a standard program.

3. Leave all parts to air-dry on a clean tea towel or drying rack.

Step 3: Disinfecting

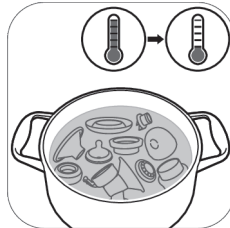
Supplies needed:

- A household pot
- Drinking-quality water

⚠ Caution: During disinfection with boiling water, prevent the bottle or other parts from touching the sides of the pot. This can cause irreversible product deformation or damage that Philips cannot be held liable for.

1. Fill a household pot with enough water to cover all parts and place all parts in the pot. Let the water boil for 5 minutes. Make sure the parts do not touch the sides of the pot.

2. Allow the water to cool down.

3. Gently remove the parts from the water. Place the parts on a clean surface/tea towel or drying rack to air-dry.

4. Store the dry items safely in a clean, protected area until needed. Ensure that the clean parts have air-dried completely before storing them to help prevent germs and mold from growing.

Using the breast pump

Cushion size

The Philips Avent single/double electric breast pump Advanced has a soft, active cushion. There is only one cushion size. It gently stimulates your nipple to trigger milk flow. The cushion is made of flexible silicone, fitting nipple sizes up to 30 mm.

Assembling the breast pump

⚠ Warning: Be careful, when you have disinfected the breast pump parts by boiling them, they can be very hot. To prevent burns, only start assembling the breast pump when disinfected parts have cooled down.

⚠ Warning: Wash your hands thoroughly with soap and water before you touch breast pump parts and breasts to prevent contamination. Avoid touching the inside of containers or lids.

Make sure you have cleaned and disinfected the parts of the breast pump that come into contact with the breast and breast milk.

Check the breast pump kit parts for wear or damage before use and replace them if necessary.

ⓘ Note: Correct placement of the white valve, cushion, silicone diaphragm and silicone tube is essential for the breast pump to form a proper vacuum.

1. Wash your hands thoroughly with soap and water.

2. Push the white valve in the pump body as far as possible.

3. Screw the pump body onto the bottle.

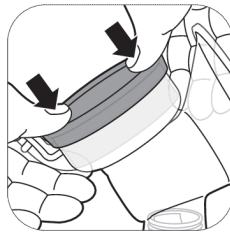
4. Place the cushion in the pump body and make sure that the rim covers the pump body.

4A. Push the inner part of the cushion into the funnel against the line (indicated with an arrow).


5. Place the silicone diaphragm in the pump body.


5A. Make sure that the silicone diaphragm fits securely around the rim by pressing it down with your thumbs.

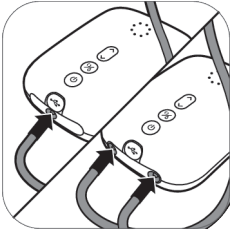6. Attach the silicone tube and cap onto the silicone diaphragm. Push down the cap until it is securely in place.

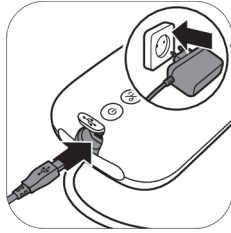7. Connect the tube(s) to the motor unit.

8. Insert the adapter into the wall socket and insert the small plug at the other end into the motor unit.

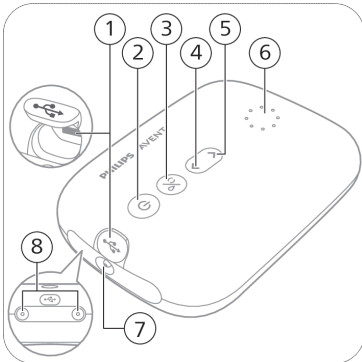
9. The breast pump is now ready for use.

Note: You can place the cover over the cushion to keep the breast pump clean when you take it with you and while you are preparing for expressing.

Motor unit part description

- 1 USB power inlet
- 2 On/off button with pause/play function
- 3 Mode selection button
- 4 'Level down' button
- 5 'Level up' button
- 6 Indicator lights for modes and levels
- 7 Tubing port for single breast pump
- 8 Tubing ports for double breast pump

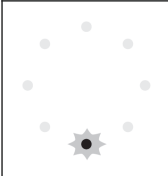
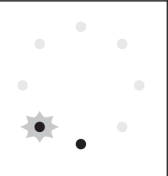
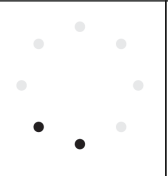
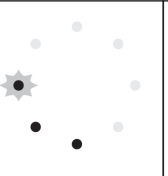
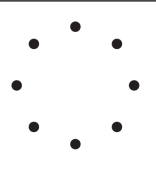
Mode indicator lights

The breast pump has two modes. Below you can find an explanation of these modes.

Indicator lights	Mode	Explanation	No. of suction levels
	Stimulation mode 	Mode to stimulate the breast to start up the milk flow.	8 suction levels
	Expression mode 	Mode for efficient milk removal after milk has started flowing. After 90 sec. of stimulation, the device automatically switches to the expression mode.	16 suction levels

Note: When you change the suction level, the display shows the selected suction level for a few seconds and then shows the selected mode again.

Suction level indicator lights

					
Suction level 1: Bottom light flashes	Suction level 2: Bottom light steady on	Suction level 3: Bottom light steady on + next light flashes	Suction level 4: 2 lights steady on.	Suction level 5: 2 lights steady on + next light flashes.	Suction level 16: all lights are on

Note: Every time the level increases, the next light starts flashing or the flashing light becomes solid.

Pumping instructions

Warning: Always switch off the breast pump before you remove the pump body from your breast, in order to release the vacuum.

Warning: Do not continue pumping for more than five minutes at a time if you do not succeed in expressing any milk. Try to express at another time during the day.

Warning: If the process becomes very uncomfortable or painful, stop using the breast pump and consult your healthcare professional.

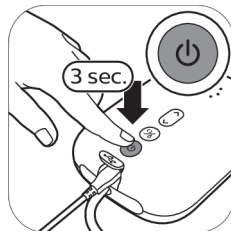
Note: If you regularly express more than 125 ml/4 fl oz per session, you can purchase and use a 260 ml/9 fl oz Philips Avent Natural bottle to prevent overflowing and spillage.


1. Wash your hands thoroughly with soap and water and make sure that your breast is clean.


2. Place the breast pump kit onto your breast. Make sure that you center your nipple.

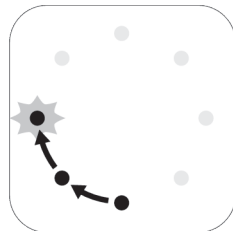
3. Press and hold the on/off button until the device switches on. The indicator lights go on to indicate that the device is in stimulation mode.

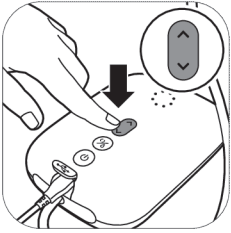
4. The breast pump starts in the stimulation mode and slowly increases its vacuum to the last used stimulation suction level (or level 5 when used for the first time).

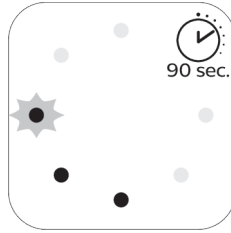
5. Use the 'level up' and 'level down' buttons to change the suction level to your preference.

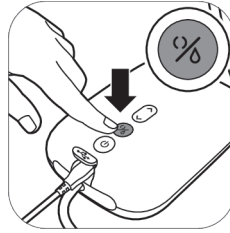
6. After 90 seconds, the breast pump will switch smoothly to expression mode and slowly increase its vacuum to the last used expression suction level (or level 11 when used for the first time).

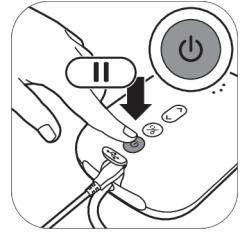
7. If your milk starts flowing earlier or if you want to switch from expression to stimulation mode, you can switch modes with the mode selection button.

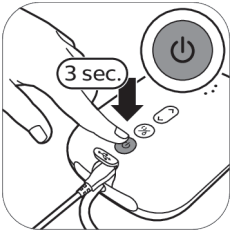8. Briefly press the on/off button if you want to pause. To continue, briefly press the on/off button again.

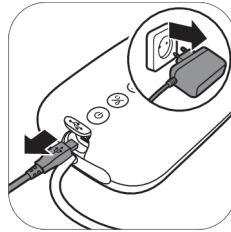9. Press and hold the on/off button to switch off.

10. Unplug the adapter from the wall socket and pull the small plug out of the motor unit.

i Tip: For optimal expression, choose the highest setting that still feels comfortable. This setting may differ across different sessions.

i Tip: You can use the double breast pump as a single breast pump by connecting only the tube of one pump kit to the motor unit.

i Note: The breast pump will switch off automatically after 30 minutes in order to prevent overheating.

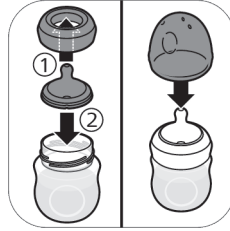
After use

1. Unscrew the bottle.

2A. To store the breast milk: close the bottle with the sealing disc and screw ring.


2B. To prepare the bottle for feeding your baby: place a teat in a screw ring and screw the assembled screw ring onto the bottle. Seal the teat with the bottle cap (see 'Assembling the bottle').

3. Remove the silicone tube and cap from the breast pump kit.

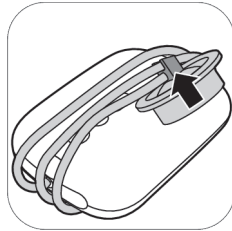4. Disassemble, clean and disinfect all parts that have been in contact with the breast and breast milk (see 'Cleaning and disinfecting').

5. For easy storage, wrap the silicone tube around the motor unit and clip the cap onto the tube.

⚠ Warning: Refrigerate or freeze expressed breast milk immediately or keep it at a temperature of 16-29 °C for a maximum of 4 hours before you feed your baby.

📄 Note: For more information about the breast pump and tips for expressing milk, visit www.philips.com/support.

Storing breast milk

Below you find guidelines for storing breast milk:

Storage location	Temperature	Max. storage duration
Room	16-29 °C	4 hours
Refrigerator	4 °C	4 days
Freezer	< -4 °C	6 months

⚠ Warning: Never refreeze thawed breast milk to avoid a decrease in milk quality.

Warning: Never add fresh breast milk to frozen breast milk to avoid a decrease in milk quality and unintended thawing of the frozen milk.

Feeding breast milk with the bottle

Choosing the right teat for your baby

Finding the right teat is important:

If your newborn baby is consistently not taking enough milk throughout the feeding sessions or has complications in getting milk, switch to a teat with a higher flow rate.

If persistent feeding issues occur, consult a healthcare professional.

Philips Avent Natural Response teats are available with different flow rates. The breast pump comes with teat number 2 (Slow).

Use a higher flow rate if your baby falls asleep during feeding, gets frustrated, or plays with the teat instead of drinking. Use a lower flow rate if your baby is gulping the milk down or if milk drips out of its mouth during feeding.

Philips Avent Natural Response teats are clearly numbered on the side, to indicate flow rate. For more information, see www.philips.com/avent.

For your child's safety and health

WARNING!

- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding.
- Throw away at the first signs of damage or weakness.
- Keep components not in use out of the reach of children.
- Never attach to cords, ribbons, laces or loose parts of clothing. The child can be strangled.
- Never use feeding teats as a soother.
- Always use this product with adult supervision.
- Keep the bottle cap away from children to avoid suffocation.
- Do not allow children to play with small parts or walk/run while using bottles.
- Inspect all parts before use and pull the feeding teat in all directions. Throw away any part at the first signs of damage or weakness.
- Always discard any breast milk that is left over at the end of a feed.

Caution: To avoid damage to the bottle

- Do not place in a heated oven, plastic can melt.
- Plastic material properties may be affected by disinfection and high temperatures. This can affect the fit of the bottle cap.
- Do not leave a feeding teat in direct sunlight or heat, or leave in disinfectant for longer than recommended because this may damage the product.

Before using the bottle

- Inspect the bottle and feeding teat before each use and pull the feeding teat in all directions to prevent a choking hazard. Throw away at the first signs of damage or weakness.
- Clean and disinfect the bottle before first use and after every use (see 'Cleaning and disinfecting').

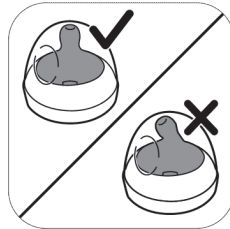
Assembling the bottle

1. The teat is easier to assemble if you move it back-and-forth while pulling it upwards instead of pulling it up in a straight line.

2. Make sure you pull the teat through until its lower part is aligned with the screw ring.

3. Make sure you place the cap vertically onto the bottle so that the teat sits upright.

4. Screw the screw ring with teat and cap on the bottle.

5. To remove the cap, place your hand over the cap and your thumb in the dent of the cap.

Heating breast milk

If you use frozen breast milk, let it defrost completely before you heat it.

Note: If you need to feed your baby quickly, you can defrost the milk in a bowl of hot water.

Heat the bottle with defrosted or refrigerated breast milk in a bowl of hot water or in a bottle warmer. Remove the screw ring and sealing disc from the bottle.

⚠ Warning: It is not recommended to use a microwave to warm up breast milk. Microwaves might alter the quality of breast milk and destroy some valuable nutrients and may produce localized high temperatures. If you heat up breast milk in the microwave, only place the container without screw ring, teat and cap in microwave. Always stir heated breast milk to ensure even heat distribution and check the temperature before serving.

Storing the bottles

- Store all parts in a dry, clean and covered container.
- Do not leave a feeding teat in direct sunlight or heat, or leave in disinfectant for longer than recommended because this may damage the product.
- For hygiene reasons, we recommend replacing the teats after 3 months.

Compatibility

The Philips Avent single/double electric breast pump Advanced is compatible with all baby bottles in the Philips Avent Natural range and Philips Avent storage cups with adapter. We do not recommend using the Philips Avent Natural Glass bottles with the breast pump.

Ordering accessories and spare parts

To buy accessories or spare parts, visit www.philips.com/support or go to your Philips dealer. You can also contact the Philips Consumer Care Center in your country.

When you order spare teats, make sure that you use a teat with the correct flow rate for your baby (see 'Choosing the right teat for your baby') and do not mix Philips Avent Anti-colic bottle parts and teats with parts of the Philips Avent Natural bottles. They do not fit and could cause leakage or other issues.

Recycling

- This symbol means that electrical products shall not be disposed of with normal household waste.
- Follow your country's rules for the separate collection of electrical products.
- Correct disposal helps prevent negative consequences for the environment and human health.

Support

For all product support, power consumption details and the EU Declaration of Conformity, please visit www.philips.com/support.

Troubleshooting

This chapter summarizes the most common problems you could encounter with the breast pump. If you are unable to solve the problem with the information below, visit www.philips.com/support for a list of frequently asked questions or contact the Consumer Care Center in your country.

Problem	Solution
I experience pain when I use the breast pump.	<ul style="list-style-type: none"> - Choose a suction level that is comfortable for you. - Check if the silicone diaphragm is not damaged (does not have small cracks, holes etc.). - Make sure that your nipple is placed in the center of the cushion. - If the pain persists, stop using the breast pump and consult your healthcare professional.
The breast pump does not switch on (indicator lights do not go on).	<ul style="list-style-type: none"> - Check if you are using the adapter that was supplied with the device and that the adapter is connected correctly to the motor unit and wall socket. - Connect the adapter to another wall socket. Then press and hold the on/off button to check if the breast pump can be switched on. - If the problem persists, contact the Consumer Care Center via www.philips.com/support.
All indicator lights on the motor unit are flashing and device shuts off automatically.	<ul style="list-style-type: none"> - Try to switch on the motor unit again. - Check if you are using the adapter that was supplied with the device. - If the problem persists, contact the Consumer Care Center via www.philips.com/support.
I do not feel any suction/the suction level is too low.	<ul style="list-style-type: none"> - If possible, try a higher suction level. - Check if the silicone diaphragm is assembled correctly (see 'Assembling the breast pump'). - Check if the white valve is not damaged and assembled correctly (see 'Assembling the breast pump'). - Check if the other parts of breast pump are intact and assembled correctly (see 'Assembling the breast pump'). - Make sure that the breast pump is positioned correctly on the breast to allow a seal between breast pump and breast. - If you still do not feel any suction, please contact the Consumer Care Center via www.philips.com/support.
The suction of the breast pump is too strong.	<ul style="list-style-type: none"> - If possible, try a lower suction level. When you express for the first time, start with the default suction level (level 5 in stimulation mode, level 11 in expression mode) and increase/decrease the level when needed. During different expression sessions, different levels may feel most comfortable for you. - Make sure that you only use the breast pump parts provided with this breast pump. - Check if the silicone diaphragm is not damaged (does not have small cracks, holes etc.). - If the problem persists, contact the Consumer Care Center via www.philips.com/support.

Problem	Solution
I express little/no milk when I use my breast pump.	<ul style="list-style-type: none"> - If you do not feel enough suction, please check Troubleshooting item 'I do not feel any suction/the suction level is too low'. - If you are using the Philips Avent breast pump for the first time, you may need some practice before you are able to express any milk. More expressing tips can be found on the Philips website www.philips.com/mybreastpump.
When I switch on the breast pump after storage, it does not start working immediately.	Perhaps you stored the breast pump at a temperature below the minimum storage temperature or above the maximum storage temperature. Let the breast pump acclimatize in a room of 20 °C for 30 minutes to adjust its temperature to the operating range (5 °C to 40 °C).

Supplemental information

Below some common breastfeeding-related conditions are described. If you experience any of these symptoms, contact a healthcare professional or breastfeeding specialist.

Common breastfeeding-related condition	Description
Pain sensation	Pain felt on the breast or nipple, or felt as a response to the suction being above a comfortable level.
Sore nipples	Persistent pain in the nipples at the beginning of the pumping session, or that lasts throughout the entire pumping session, or pain between sessions.
Engorgement	Swelling of the breast. The breast may feel hard, lumpy and tender. May include erythema (redness) of the breast area and fever. Usually occurs during the first days of lactation.
Bruise	A reddish-purple discoloration that does not fade when pressed. When a bruise fades, it may become green and brown.
Thrombus	Thrombus can result into tenderness, pain, redness, swelling and/or heat.
Injured tissue on nipple (nipple trauma)	<ul style="list-style-type: none"> - Fissures or cracked nipples. - Skin tissue peeling off the nipple. Normally occurs in combination with cracked nipples and/or blisters. - Nipple tear. - Blister. Looks like small bubbles on the surface of the skin. - Bleeding. Cracked or torn nipples can lead to bleeding of the affected area.
Clogged mammary ducts	A red, tender lump on the breast. May include erythema (redness) of the breast area. Can lead to mastitis (breast inflammation) and fever if left untreated.

Storage conditions

Keep the breast pump out of direct sunlight as prolonged exposure may cause discoloration. Store the breast pump and its accessories in a safe, clean and dry place.

If the device has been stored in a hot or cold environment, place it in an environment with a temperature of 20 °C for 30 minutes to let it reach a temperature within the usage conditions (5 °C to 40 °C) before you use it.

Electromagnetic fields (EMF)

This Philips device complies with all applicable standards and regulations regarding exposure to electromagnetic fields.

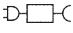
Technical information


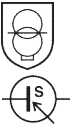
	Single electric breast pump	Double electric breast pump
Vacuum level	Stimulation: -60 to -200 mbar (-45 to -150 mmHg) Expression: -60 to -360 mbar (-45 to -270 mmHg) Adjustable in steps of 20 mbar Intermittent vacuum	
Cycle speed	53-120 cycles/min.	
Motor unit rated input	5 V direct current / 1.1 A	5 V direct current / 1.8 A
Mains adapter rated input	100-240 V  50/60 Hz 400 mA	
Adapter type number	S009AHz050yyyy The letters "yyyy" represent the output current from 0100 (1000 mA) to 0180 (1800 mA), increments in steps of 100 mA. The letter "z" denotes type of plug, it can be: U, V, B, S, C, A, D, K, T, I, E, F.	
Mains adapter rated output	5 V direct current / 1.1 A	5 V direct current / 1.8 A
Operating conditions	Temperature: 5 °C to 40 °C Relative humidity: 15 to 90 % (non-condensing)	
Storage and transportation conditions	Temperature: -25 °C to 70 °C Relative humidity: up to 90 % (non-condensing)	
Operating and storage pressure	700-1060 hPa (<3000 m altitude)	
Net weight	230 g	310 g
External dimensions	Motor unit: 145 mm x 95 mm x 45 mm (length x width x height)	
Device classification	IEC 60335-1: System: Class II; motor unit: Class III Construction	
Materials	ABS, silicone (motor unit) Polypropylene, silicone (other parts)	
Service life	500 hours	
Mode of operation	Continuous operation	
Automatic power-off	After 30 minutes (after last user interaction)	
Ingress protection	IP22 (motor unit), IP20 (adapter)	

Explanation of symbols

The warning signs and symbols are essential to ensure that you use this device safely and correctly and to protect you and others from injury. Below you find the meaning of the warning signs and symbols on the label and in the user manual.

Symbol	Explanation
	Indicates to follow the instructions for use.
	Indicates important information such as warnings and cautions.
	Indicates usage tips, additional information or a note.
	Indicates the manufacturer.
	Indicates the manufacturing date.
	Indicates that the manufacturer has taken all necessary measures to ensure that the device complies with the applicable safety legislation for distribution within the European Union.
	Indicates manufacturer's catalog number of the breast pump.
	Indicates the batch number of the breast pump.
	Indicates that the part of the appliance that comes into physical contact with the user (also known as the applied part) is of type BF (Body Floating) according to IEC 60601-1. The applied part is the breast pump kit.
IP22	Indicates the protection against ingress of solid foreign objects greater than 12.5 mm and against harmful effects due to the ingress of dripping water when tilted at 15 degrees.
IP20	Indicates the protection against ingress of solid foreign objects greater than 12.5 mm and no protection against ingress of water.
	Indicates 'alternating current'.
	Indicates 'direct current'.
	Indicates 'Class II Equipment'. The adapter is double-insulated (Class II).

Symbol	Explanation
	Indicates USB.
	Indicates the storage and transportation temperature limits to which the device can be safely exposed: -25 °C to 70 °C.
	Indicates the relative humidity upper limits to which the device can be safely exposed: up to 90 %.
	Indicates 2-year Philips worldwide guarantee.
	Indicates Forest Stewardship Council. The FSC trademarks enable consumers to choose products that support forest conservation, offer social benefits, and enable the market to provide an incentive for better forest management.
	Indicates that electrical products shall not be disposed of with normal household waste.
	Indicates that the product should not be discarded as unsorted waste but must be sent to separate collection facilities for recovery and recycling.
	Indicates that a specific adapter is required for connecting the breast pump.
	Indicates that the adapter meets the requirements of the applicable India safety standards.
	Indicates that this product has been certified by the Bureau of Indian Standards.
	Indicates that the breast pump meets the requirements of the UK regulations.
	Indicates that the material used in the product is safe for food contact.
	This logo confirms that the product complies with Ukrainian technical regulations.
	This logo confirms that the product complies with Serbian technical regulations.
	Indicates that the breast pump meets the requirements of the Argentina regulations.
	Indicates that the material used for the packaging is corrugated board.

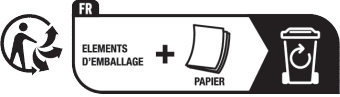Symbol	Explanation
	Indicates that the material used for the packaging is non-corrugated cardboard/fiberboard.
	Indicates that the material used for the plastic bag is low density polyethylene.
	Certification mark for Australia and New Zealand that shows that this product conforms with relevant safety and Electromagnetic Compatibility (EMC) requirements.
	Indicates indoor use only.
	Indicates that the appliance is intended to be usable up to the maximum altitude of 3000 m.
	Indicates that the separate power supply unit (adapter) contains a short-circuit-proof safety isolating transformer.
	Indicates that the power supply unit shall not be used if pins of the plug part are damaged.
	Indicates the maximum temperature at which the separate power supply unit (adapter) may be used continuously under normal operating conditions.
	Indicates that the part is certified with six level energy efficiency rating.
	Indicates that the product should not be used after the period accompanying the symbol. The symbol indicates the expiry date of x years from the manufacturing date (FAB).
	Indicates that the paper packaging should be deposited in the blue container.
	Indicates that the plastic bags should be deposited in the yellow container.
 <p>FR ELEMENTS D'EMBALLAGE + PAPIER</p>	Indicate that the product or packaging needs to be sorted or brought to a recycling point.
 <p>FR Cet appareil et ses accessoires se recyclent</p> <p>A DÉPOSER EN MAGASIN OU A DÉPOSER EN DÉCHÈTERIE</p> <p>Points de collecte sur www.quefairedemesdechets.fr</p>	

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Trademarks are the property of Koninklijke Philips N.V. / Les marques commerciales sont la propriété de Koninklijke Philips N.V. / I marchi commerciali e depositati sono di proprietà di Koninklijke Philips N.V.
All rights reserved. / Tous droits réservés. / Tutti i diritti riservati.
Philips Consumer Lifestyle B.V.