Cat.no. 78/021 ms001

GB • The cooling pad after childbirth acts as a cold compress, relieving pain and providing immediate relief in intimate areas, both after physiological childbirth and after a cesarean section. Additionally, the pad is absorbent, thanks to which it absorbs postpartum secretions, simultaneously acting as a cooling compress and pad in one product. Do not use in the case of injuries with inflammation. Retain the instruction for later use.

Warning

- The product is intended for external use only.In the event of contact of the liquid with eyes or skin, rinse the area with clean water.
- Do not pierce the packaging with sharp objects or apply excessive pressure to it. Do not hit or place anything on the packaging.
- If the packaging is damaged, do not use it.
- The product should not be consumed.
- Do not use cryotherapy for longer than 20 minutes.
- Do not refrigerate or freeze for reuse.
- Do not use on injuries where there is still inflammatory exudate after 24 hours. Do not use on people with sensory disorders.
- Keep out of reach of children.
- If you have circulatory problems, consult a doctor before use.
- Stop using the product if it causes discomfort.
- If you have any concerns about your health, consult a doctor.The Canpol babies cooling pad is a disposable product with a cold activator that starts the cooling effect immediately after its activation. It is completely safe to use and its use is not associated with the risk of frostbite.

How to use?

Wash your hands and holding it with both hands, squeeze the place where you feel the lump. Don't be afraid to press hard to break through the special granulate, which will turn into liquid under pressure. Shake the pad a few times to distribute the liquid evenly, which activates the cooling effect when it gets inside the pad. Then remove the pad from the protective film, remove the adhesive strip and place it in your underwear. You will feel the cooling effect immediately, but the most intense cooling sensation will occur after about 2 minutes. The pad will remain cold for about 30 minutes. Throw the used pad in the trash.

Storage

The product should be stored in a shaded and dry place away from heat sources, protected from direct sunlight, do not mix with chemical products and do not excessively crush or freeze.

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